

K-ASSAY®

Urine FDP Sample Diluent

Cat. No. KAI-109D**INTENDED USE**

The **K-ASSAY®** Urine FDP Sample Diluent is intended to be used for the dilution of urine with the **K-ASSAY®** Serum/Urine FDP Assay and the **K-ASSAY®** Serum/Urine FDP-E Assay. **FOR RESEARCH USE ONLY IN THE U.S.** Not for use in diagnostic procedures in the U.S.

SUMMARY

This is to be used for the dilution of high urine samples for the **K-ASSAY®** Urine/Serum FDP Assay when necessary (i.e. when the concentration is greater than the highest calibrator, approximately > 3.2 µg/mL), and/or for the dilution of high urine samples for the **K-ASSAY®** Serum/Urine FDP-E Assay when necessary (i.e. when the concentration is greater than the highest calibrator, approximately > 192 ng/mL).

KIT COMPOSITION

Urine FDP Sample Diluent 1 x 40 mL
saline solution with 0.05% sodium azide

WARNINGS AND PRECAUTIONS

FOR RESEARCH USE ONLY IN THE U.S. Not for use in diagnostic procedures in the U.S.

The diluent in this kit contains sodium azide as a preservative. Sodium azide may form explosive compounds in metal drain lines. When disposing of calibrators through plumbing fixtures, flush with copious amounts of water. For further information, refer to "Decontamination of Laboratory Sink Drains to Remove Azide Salts," in the Manual Guide-Safety Management No. CSC-22 issued by the Centers for Disease Control, Atlanta, Georgia.










STORAGE AND HANDLING

Product should be stored refrigerated (2-8°C). Return to 2-8°C promptly after use. Unopened diluent can be used for 2 years from the date of manufacturing as indicated by the expiration date on the package and bottle labels. Opened diluent can be used up to the expiration date provided that the cap is tightly closed to prevent evaporation and the solution is not contaminated.

PROCEDURE

Diluent is ready to use.

LABELING SYMBOLS

	Lot Number
	Reagent
	Expiration or "Use By" Date
	Catalog Number
	2-8°C
	Temperature Limitation. Store between 2 and 8 degrees C
	Manufacturer
	Consult Package Insert for Instructions for Use
	Authorized Representative in the European Community

EU AUTHORIZED REPRESENTATIVE**EC REP****Advena Ltd.**

Tower Business Centre, 2nd Flr.,
Tower Street, Swatar, BKR 4013 Malta

ORDERING / PRICING / TECHNICAL INFORMATION**KAMIYA BIOMEDICAL COMPANY**

12779 Gateway Drive
Seattle, WA 98168 USA
TEL: (206) 575-8068 / (800) 526-4925
FAX: (206) 575-8094